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Instruction Manual

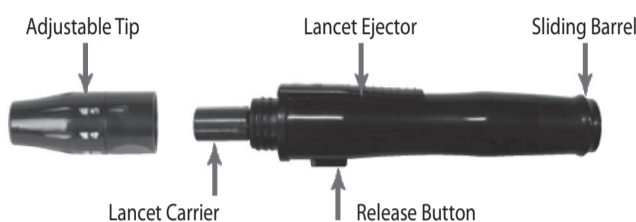
1. Intended Use

Glucosefine® lancing device is a convenient medical tool for collecting capillary blood samples with Glucosefine® lancets for glucose monitoring or other tests which require one or two drops of blood.

2. ⚠ Caution

- 1) This device is intended for use always on the same person. Having other persons use an individual's device may lead to contamination.
- 2) If the lancing device is to be used by another person, the unit must be provided with a new tip and lancet.
- 3) For safety reasons and in order to avoid accidental pricks, do not leave the lancet in the device after use.
- 4) The new lancet must be put into the lancing device only right before testing.
- 5) The device has small parts which if ingested can pose a choking hazard to children.
- 6) In case of use in medical practice, hospital or nursing care (POC) users need to consult their own infection control protocols in order to avoid any contamination or infection.

3. Product Overview and Features

4. Glucosefine® Sterile Lancet:

Glucosefine® lancing device is intended to be used only with Glucosefine® Sterile Lancets.

Important Notes for Use:

BEFORE use: Twist off the protective cap (1) and pull it (2).

AFTER use: Push lancet into flat side of protective cap and discard properly.

CAUTION! Do not use in case of a damaged or missing protective cap!

Lancets must not be reused and have to be disposed of in an appropriate container. Your pharmacy will give you further information concerning appropriate disposal.

Glucosefine® lancets are intended to be used only with Glucosefine® Lancing Device!

Lancet can pose a choking hazard to children if swallowed.

Storage:

Lancets shall be stored at room temperature and protected against sunlight.

5. Operational Instructions

How to use Glucosefine® lancing device with adjustable tip

Before taking a blood sample: Wash your hands carefully with warm water to increase the circulation of blood in the fingers. Dry hands thoroughly and make sure the finger to be pricked is completely dry.

- 1 Unscrew the tip of the lancing device by turning it anti-clockwise.
- 2 Insert a new Glucosefine® lancet into the bottom of the lancet carrier.
- 3 Twist off the protective cap of the lancet. Put back the tip of the lancing device and turn it clockwise.
- 4/5 To cock the lancing device, pull the sliding barrel backwards until a „click“ sound is heard.
- 6 Select a penetration depth depending on skin type. The tip features 5 depth settings: setting 1 is appropriate for delicate skin, like children's, while setting 5 is more suitable for thick skin. When using the lancing device for the first time, select a setting in-between (setting 3).
- 7 Place the lancing device on the side of your finger tip and press the release button. After using the blood sample unscrew the tip of the lancing device again by turning it anti-clockwise.
- 8 Push the lancet ejection button forward while pulling the sliding barrel backwards. This will eject the lancet. Discard the used lancet properly, for example in a biohazard container.

Optional clear tip for Alternate Site Testing (AST)

By request Glucosefine® lancing device is optionally supplied with a clear tip which allows to collect blood samples from other sites than the fingertip, like forearm and belly. Please absolutely read instructions for AST in the user's manual of your measuring system. If there are no instructions in that manual contact your physician before taking blood samples from alternate sites.

6. Storage, Cleaning and Disinfection

1) Storage:

Products shall be stored at room temperature and protected against sunlight.

2) Cleaning and Disinfection:

It is recommended that you clean and wipe the outside of Glucosefine® lancing device minimum once a week with a soft cloth dampened with mild soap and water. Wash the adjustable tip and the AST tip once a week with mild soap. Do NOT immerse in liquid. Place it on paper or towel and let it dry out.

7. Warranty

Glucosefine® lancing device has 2 years warranty period from the date of purchasing. A malfunctioning device should be returned to METRADO GmbH or the local representative in your country. In case of warranty claim we replace your lancing device for free.


8. Specifications

Glucosefine® Lancing Device				
Product	REF	PZN	Overall Length	Max. Ø
Glucosefine® Lancing Device	4298004	02490161	106,4 mm	16 ± 0.1mm
Material : Polycarbonate, ABS - Color: black				

Glucosefine® Sterile Lancet				
Product	REF	PZN	Overall Length	Needle
Glucosefine® Sterile Lancet	4226005	02491479	32 mm	28G

9. Symbol and Label Reference

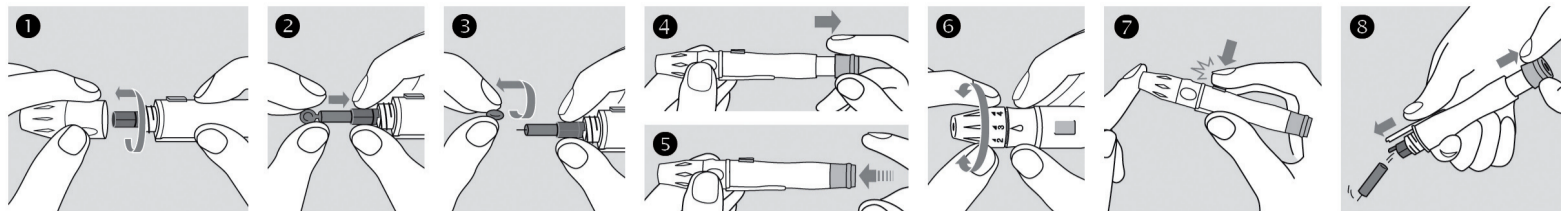- Attention - See instructions for use!
- Consult Instruction Manual for Use!
- Do not reuse
- LOT number
- Reference number
- Manufacturer
- Date of Manufacture
- 0483 This Product Fulfills the Requirements of Directive 93/42/EEC on Medical Devices.

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