# Glucoseo*fine*®

Blutzucker-Mess-System | Blood Glucose Monitoring System

Système de mesure de la glycémie | Sistema di monitoraggio della glicemia

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# Dear Glucofine® Blood Glucose Monitoring System Owner

Thank you for choosing the Glucofine® Blood Glucose Monitoring System. This booklet has important information you must know about the Glucofine® Blood Glucose Monitoring System. Please read it carefully. We understand that self-testing of blood glucose level provides a way to control your diabetes and may give you peace of mind by testing regularly. As a result Glucofine® was developed to provide you with a fast and accurate reading with a convenient and simple process. Our goal at Glucofine® is to provide the best quality healthcare products coupled with superior customer service. Always consult with your healthcare professional before making any changes to your diabetes management. The Glucofine® Blood Glucose Monitoring System is for in vitro diagnostic use only.

## Important information

The Glucofine® Blood Glucose Monitoring System is intended for use outside the body (in vitro diagnostic use only). It should be used only for testing blood glucose and only with fresh capillary whole blood samples. It should not be used for the diagnosis of diabetes.

Consult your physician or diabetic healthcare professional about daily management of your diabetes and proper use of the glucometer.

Please pay close attention when handling blood. Improper procedures may cause serious hazards to your health.

The Glucofine® Blood Glucose Monitoring System contains small parts. Please keep your monitoring system out of reach of children.

## Test Principle

Glucose in the blood sample will react to the electrodes in the test strip, generating an electrical current that will stimulate a chemical reaction. This reaction is measured by the Glucofine® meter and displayed as your blood glucose result.

The Glucofine® Blood Glucose Monitoring System is plasma-calibrated to allow easy comparison of results with laboratory methods.

Blood Glucose Monitoring Systems which are calibrated against a whole blood method may have different results in comparison to Glucofine® Blood Glucose Monitoring System.

The laboratory system used for calibration of the Glucofine® Blood Glucose Monitoring System is YSI 2300 STAT plus which is equipped with a glucose oxidase system.

### Note

#### < Maintaining the Glucofine® Blood Glucose Meter >

1. Do not assemble or disassemble the Blood Glucose Meter.
2. Store your Blood Glucose Meter in a dry place.
3. Avoid getting dirt and moisture on your Blood Glucose Meter.
4. Do not store the Glucofine® meter in places where chemicals or gas may occur.
5. Keep out of direct sunlight.

# Contents

1. Learning the System	
Intended use of Glucofine® Blood Glucose Test Meter	51
Glucofine® Blood Glucose Monitoring System: Kit Contents	52
Glucofine® Blood Glucose Test Meter	53
Glucofine® Blood Glucose Test Meter: Display	54
Glucofine® Blood Glucose Test Strip Information	55-56
2. Before Testing	
Setting Your Blood Glucose Test Meter	57-58
Auto-Coding Function	59
Backlight Function	60
Performing a Glucose Control Solution Test	61-62
3. Testing Your Blood Glucose	
Preparation	63
Collecting a Drop of Blood	64-66
Testing Procedure	67-70
Test Strip Ejector Function	71
4. Reviewing Your Results	72-73
5. Deleting Test Results	74
6. Alternate Site Testing	75
7. Performing Blood Tests in the Alternate Sites	76-77
8. Glucofine® Software	78
9. Taking Care of Your Blood Glucose Test Meter	
Cleaning Your Meter and Maintenance	79
Replacing the Batteries	80
10. Troubleshooting	81-83
11. Specifications	84
12. Conversion table mmol/L <--> mg/dL	85
13. Warranty	86
14. Symbol Reference	87

## Intended use of Glucofine® Blood Glucose Test Meter

The Glucofine® Blood Glucose Monitoring System is intended for in vitro diagnostic use only. Glucofine® Blood Glucose Monitoring System is for self-testing.

It should not be used for the diagnosis of diabetes or for the testing of newborns.

The Glucofine® was designed to test a blood glucose (sugar) level and can be used with a capillary whole blood sample.

Do not use the Glucofine® Blood Glucose Monitoring System for any other than for blood glucose testing.

### CAUTION

Before using any product, please read all instructions.

Consult your physician or diabetic healthcare professional about daily management of your diabetes and proper use of the glucometer. If you have any questions about the Glucofine® product, please contact your local representative.

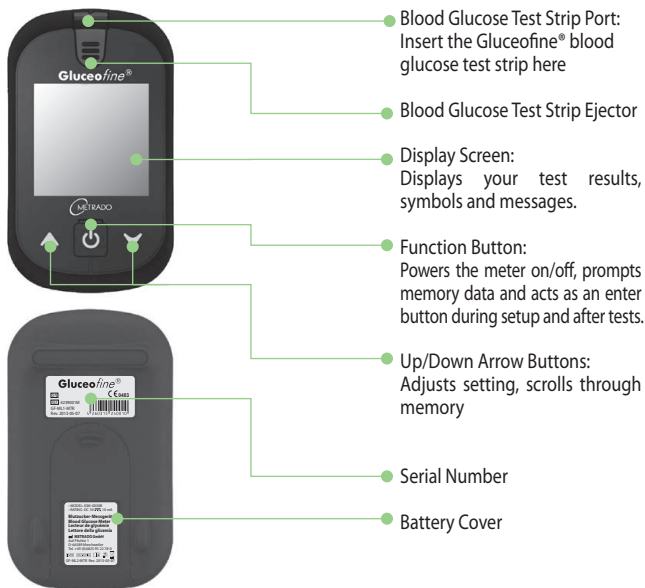
# Glucosefine® Blood Glucose Monitoring System: Kit Contents

1. Glucosefine® Blood Glucose Test Meter
2. Glucosefine® Blood Glucose Test Strips
3. Lancing Device
4. Lancets
5. Operation Manual
6. Warranty Registration Card
7. Patient Logbook
8. Carrying Case
9. 2 Spare Batteries  
(3V Li-CR2032)

Your Glucosefine® Blood Glucose Monitoring System has been sealed and it should contain all of the parts shown above. Please return the system to place of purchase or contact your nearest Glucosefine® representative if the container seals are broken or parts are missing.


# Glucosefine® Blood Glucose Test Meter

## CAUTION

- 1) Do not use Glucosefine® blood glucose test meter close to synthetic materials. Synthetic clothes, carpets etc. may cause damaging static discharges in a dry environment.
- 2) Do not use Glucosefine® blood glucose test meter near cellular or cordless telephones, walkie talkies, garage door openers, radio transmitters, or other electrical equipment that are sources of electromagnetic radiation, as these may interfere with the proper operation of the blood glucose test meter.
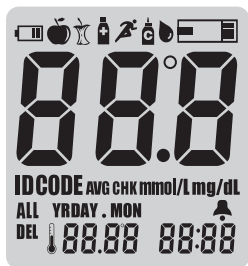
# Glucosefine® Blood Glucose Meter Display

 Low Battery Warning Symbol

 Before Having a Meal

 After Having a Meal

 After Taking Medication

 After Sports Activity

 Control Solution Symbol

 Blood drop symbol for test

**ID** User ID Symbol

**CODE** Strip Code Symbol

**AVG** Average Test Result System

**CHK** Troubleshooting Symbol

**mmol/L mg/dL** Test Result Unit Symbol

**ALL  
DEL** Delete results

 Alarm Symbol

**YRDAY . MON**  
 **88.88** Temperature / Date

**88:88** Time / Test Result Amount

# Glucosefine® Blood Glucose Test Strip Information

## Storage and Handling Caution

1. Keep out of direct sunlight.
2. Store the Glucosefine® Blood Glucose Test Strip vials in a cool and dry place between 2 - 30 °C (36 - 86 °F). Do not freeze.
3. Make a notation of the discard date on the vial label when you first open it. Discard remaining Glucosefine® blood glucose test strips 6 months after first opening the vial.
4. Close the vial cap tightly and immediately after removing a Glucosefine® blood glucose test strip.
5. Do not use test strips after the expiry date printed on the package or vial since it may cause inaccurate results.
6. Avoid getting dirt, food or water on the blood glucose test strip. Do not bend, cut, or alter the Glucosefine® blood glucose test strip.
7. Glucosefine® blood glucose test strips are for single use only.
8. Keep the blood glucose test strip vial away from children.
9. Please refer to the user's manual for Glucosefine® blood glucose test strip for additional information.
10. Discard the used blood glucose test strip carefully, to prevent any infection.

**Use only Glucosefine® brand blood glucose test strip with your Glucosefine® blood glucose test meter.**

# Glucosefine® Blood Glucose Test Strip Information

## Limitations of Blood Glucose Measuring System :

The blood glucose test Strips provide accurate results when the following constraints are observed :

- Use the Glucosefine® blood glucose test strips with the Glucosefine® Blood Glucose Meter.
- Use fresh capillary whole blood only.
- Do not use neonate samples.
- The blood glucose test strips are for single use only. Do not reuse.
- Dehydration may lower test results.
- Inaccurate results may occur when in shock, hypotensive, hyperglycemic, or hyperosmolar state, with or without ketosis.
- Blood glucose test strips used above altitudes of 10,000 feet will have an effect on test results.

Doctors and healthcare professionals should note the following interferences that may affect test results :

- Glucosefine® test strip results can be used with hematocrit levels in the range of 30% to 55%. If the hematocrit level is out of the range (30% to 55%), then the test results may be lower or higher than actual value. If you do not know your hematocrit level, consult your healthcare professionals.
- Interferences: Acetaminophen, Uric and Ascorbic acid (Vitamin C), and other reducing substances when occurring in normal blood or normal therapeutic concentrations do not significantly affect results. However, abnormally high concentrations may cause cause inaccurately high results.
- Lipemic samples: Cholesterol up to 510 mg/dL or triglyceride up to 3400 mg/dL do not significantly affect the results. Values beyond these levels should be interpreted with caution.
- Blood samples that contain a high concentration of dissolved oxygen may lower the test result.

## Performance Characteristics :

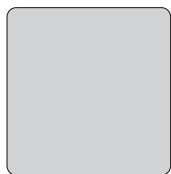
The performance of the blood glucose test strips has been evaluated in laboratory and in clinical tests. For more information see test strip leaflet.

*Measurement Range:* The measurement range of the Glucosefine® Blood glucose monitoring system is 10 to 600 mg/dL (0.6-33.3 mmol/L).

# Setting Your Blood Glucose Test Meter

The Glucofine® Blood Glucose Test Meter has a variety of excellent functional modes; beep sound to alert when function is set, date, time, five daily alarms, unique user ID, ability to designate activities related to the test result, and the ability to designate three unique data averages.

## Power On

Press  for 5 seconds at least.

## User Activity Option

After pressing and releasing either  or  buttons to turn the user Activity option on/off, confirm with .

## User ID Option

After pressing and releasing either  or  buttons to turn User ID option on/off, confirm with .

ENGLISH

## Year

After pressing and releasing either  or  buttons to set the year (2012 - 2099), confirm with .

## Date / Time

After pressing and releasing either  or  buttons to set the date and time, confirm each with .

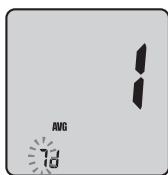
## Test Unit

Press and release either  or  buttons to set the test unit (mg/dL or mmol/L), confirm with .

## Setting Your Blood Glucose Test Meter


Number of days for average  
After pressing and releasing either ▲ or ▼ buttons to set the number of days for average calculation (3 different possible), confirm with ⏻.

Alarm on/off  
After pressing and releasing either ▲ or ▼ buttons to turn the alarm on/off, confirm with ⏻.

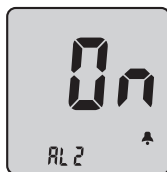Alarm time  
After pressing and releasing either ▲ or ▼ buttons to set the alarm hour and minute, confirm with ⏻.

Alarm  
After programming the number of desired alarms (5 alarms possible), confirm with ⏻.

### CAUTION

1. **VERY IMPORTANT:** Correct test unit.

Your Glucofine® blood glucose test meter is pre-set to mg/dL or to mmol/L as test unit. A wrong test unit will lead to wrong test result display! Please consult your healthcare professional if you're not sure which test unit to use. Stored test results are displayed correctly after re-setting the test unit. See also conversion table on page 85.

2. Without setting the date properly, the average glucose level and the results in memory will not show proper values. It is recommended to set the blood glucose test meter before use and when new batteries are installed.

3. You can't test your blood glucose in the setting mode.

4. To turn the meter off during the setup, press ⏻ for at least 3 seconds.

## Auto-Coding Function

1. Glucofine® has automatic code recognition function.

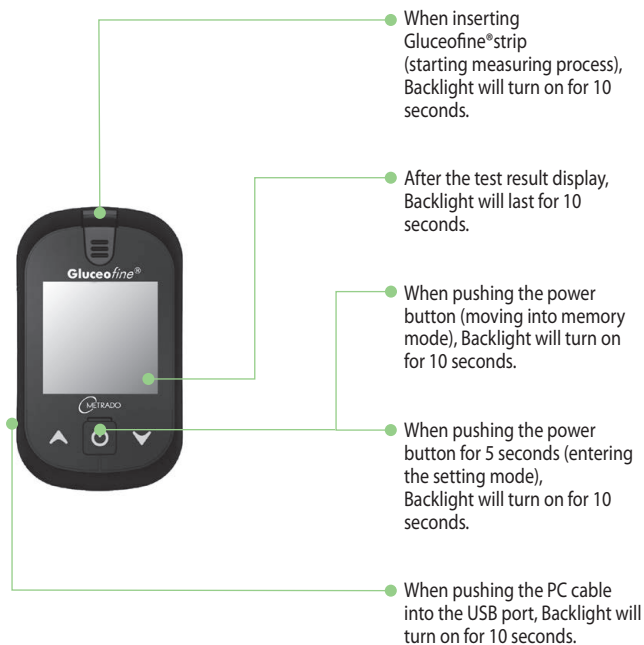
2. It recognizes automatically the code number. This is a very convenient function preventing the inconvenience of setting the code number on the blood glucose test meter every time.

### CAUTION

1. Do not bend the blood glucose test strip to prevent the automatic code recognition failure.
2. If the code recognition label is damaged, the code recognition may fail. Please check the code number on the LCD window with the code number on the blood glucose test strip vial.
3. If you do not check the code number, pull the blood glucose test strip out, conduct measuring procedures from the beginning.
4. If the code on the LCD window and on the blood glucose test vial does not match, repeat measuring with a new test strip. If the problem persists, you should contact your local distributor (see last page for contact).

## Backlight Function

## Performing a Glucose Control Solution Test

The glucose control solution should be used when there is suspicion that the blood glucose test meter or the blood glucose test strips are not working properly. For example: if strips have been exposed to extreme conditions or if you drop the blood glucose test meter.

Check the expiration date before performing a glucose control solution test.

Do not use if expired. Once opened, solution expires after 3 months.

The glucose control solution should be used

- Whenever you suspect the blood glucose test meter or blood glucose test strip is not functioning properly.
- If your blood glucose test results are not consistent with your symptoms or if you do not think they are accurate.
- If you have dropped the blood glucose test meter.
- For quality control in the point of care usage.
- For teaching or learning the system.

1. Shake the glucose control solution before use.
2. Discard the 1st drop of glucose control solution, this will eliminate any residue.
3. Place a drop of glucose control solution on a clean dry surface (ex. The lid of the blood glucose test strip vial).
4. Insert a blood glucose test strip into the port and press the up arrow. A glucose control solution vial will appear on the upper part of the screen, above the date and time. This will allow you to differentiate between a glucose control solution test and an actual blood test for future reference.
5. Dip the blood glucose test strip into the glucose control solution.
6. Result appears in 5 seconds.
7. Compare the result to the range printed on the blood glucose test strip vial, the results should fall within that range.
8. If your glucose control solution falls out of range, do not perform a glucose test, please contact Metrado GmbH or your local representative.

## Testing Your Blood Glucose - Preparation

Before testing, be sure to read this section and blood glucose test strip instruction leaflet found in the blood glucose test strip box carefully.

Make sure you have all the necessary items to begin testing:

- Glucofine® blood glucose test meter
- Glucofine® blood glucose test strip
- Lancing Device
- Sterile Lancets

### CAUTION

To reduce risk of infection :

1. The lancing device and sterile lancets should NOT be shared with others.
2. Always use a new, sterile lancet and a new blood glucose test strip.
3. Practice using the Lancing Device and become accustomed with its use.
4. Wash your hands with warm, clean water and dry completely before testing.
5. Avoid getting hand lotion, oils, dirt or debris on the lancets or on the lancing device.

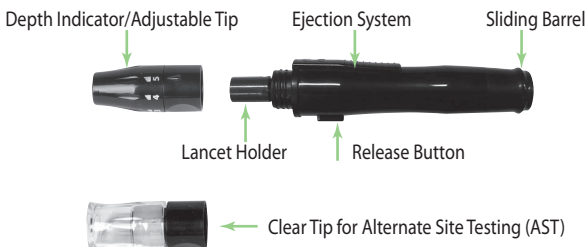
# Collecting a Drop of Blood

The lancing device is a convenient medical tool for collecting capillary blood samples for glucose monitoring or other tests which require one or two drops of blood. As a more advanced lancing device, it provides ultimate safety and comfort for obtaining blood samples. An adjustable tip offers 5 levels of skin penetration for individual user's comfort and a lancet ejector enables the safe disposal of the used lancet. As an added bonus, the lancing device also comes with a transparent tip for AST (Alternative Site Testing).

## PRECAUTIONS

- Never use a lancet that has been used by someone else. This could lead to contamination.
- If the lancing device is to be used by another person, the unit must be properly disinfected and a new tip and lancet must be used.
- Do not leave the lancet in the device after use.
- A new lancet must only be placed into the lancing device directly before testing.
- This device has many small parts and could be a choking hazard for children if swallowed.
- In the case of use in medical practice, hospital or nursing care (POC) users need to consult their own infection control protocols in order to avoid any contamination.
- Always dispose of the used lancet in a biohazard garbage container.

## COMPONENTS


## Collecting a Drop of Blood

1. Unscrew the tip of the lancing device by turning it counter-clockwise while holding the base firmly.

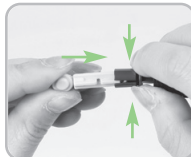2. With two fingers, pull out the lancet carrier and hold it in place.

3. While holding the lancet carrier, insert a new sterile lancet into the bottom of lancet carrier.

4. Twist the cap of the lancet off.

5. Put the tip of the lancing device back on and turn it clockwise. By turning the front part you can adjust 5 penetration depths.

6. Hold the tip firmly in one hand then pull out the sliding barrel with the other hand. This will cock the lancing device.

7. Place the lancing device on the side of your finger. Press the release button to take a sample.

8. Unscrew the tip of the lancing device by turning it counter-clockwise.

9. Push the lancet ejector forward with the thumb and simultaneously pull out the sliding barrel to dispose of the used lancet in a proper biohazard container.

# Collecting a Drop of Blood

## STERILE LANCETS

Glucoseofine® lancing device is intended to be used only with sterile lancets Glucoseofine® or infopia®. Lancets can be purchased in your pharmacy or with your local distributor.

The Monitoring System comes with an additional clear tip for the Glucoseofine® Lancing Device. It can be used with the lancing device instead of the black tip for collecting blood samples from alternate sites. Please refer to pages 75-77.

## STORAGE

Lancing Device and Sterile Lancets must be stored at room temperature protected from sunlight or moisture.

## CLEANING AND CARE

Clean and wipe the outside of the lancing device a minimum of once a week with a soft cloth, dampened with mild soap and water. Wash the adjustable tip and the clear tip once a week with mild soap. DO NOT immerse in liquid.

## WARRANTY

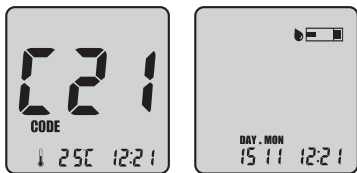
Lancing device has a 2-year warranty from the date of purchase. A malfunctioning device should be returned and will be replaced for free in case of warranty claim.

## Testing Procedure

### Step 1

**The meter must be turned OFF before inserting the test strip.**

Firmly insert the test strip into the meter test port. Insert down firmly in direction of arrow on test strip (arrow up). Please do not insert the test strip upside down. When you insert the test strip into the meter, the power automatically turns on with the code and temperature. After a while the code number and testing temperature will disappear. When the strip sign blinks, apply your blood to the test port of strip.

### CAUTION

1. If the code is not displayed after for a while, pull the blood glucose test strip out of the port, and re-start the procedure from the beginning.
2. If the code on the LCD display window and on the test vial does not match, try another new blood glucose test strip. If the mismatch persists, please contact your local representative for help.
3. If the blood glucose test meter does not power on, pull the blood glucose test strip out of the port, and re-insert the blood glucose test strip.
4. Avoid testing under direct sunlight, for a more accurate test result.
5. If you apply your blood sample too early, Er5 message will appear on LCD ( refer to page 81).

## Testing Procedure

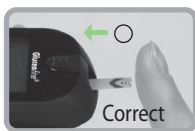
### Step 2

- Place the top edge of the blood glucose test strip to the area you prepared for testing.
- The blood will automatically be “drawn” into the blood glucose test strip channel.
- Do not take off your finger until you hear a beep sound.

#### • Correct


Completely filled

#### • Incorrect

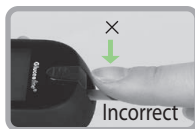Poorly filled

- Glucofine® blood glucose test meter only requires 0.5 $\mu$ l sample volume.


### CAUTION

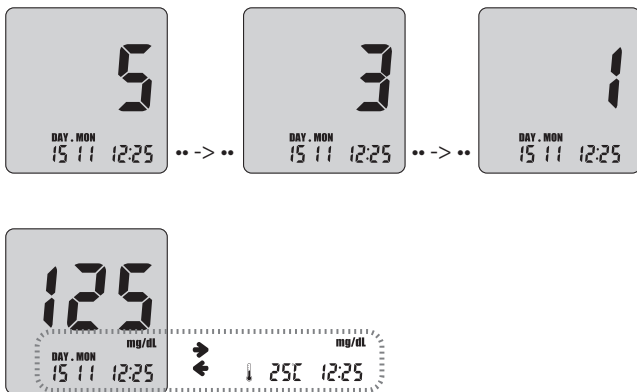1. Make sure that the confirmation window is completely filled with blood!
2. If you have a poor amount of blood, please run a new test with a NEW test strip.
3. If you do not conduct the test within 3 minutes, the blood glucose test meter will automatically power off to save battery life. In this case start a new test procedure.

## Testing Procedure

### Step 3

- After the beeping sound, the test will begin automatically and your results will appear in 5 seconds. It should begin counting down from 5 to 1 second on the LCD display window. The LCD window will display the result of your blood glucose level, temperature and time.
- Record the result value in your logbook. When blood glucose test strip is removed, the blood glucose test meter turns off automatically.

### CAUTION

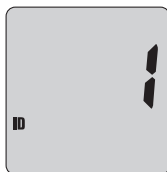
1. If the test result is out of the test range, the Hi/Lo message will be shown on the LCD.
2. Safely discard used blood glucose test strip and lancet, to a proper place, according to the local regulation.

## Testing Procedure with activated User Activity Option and User ID.

### Step 4-1

- If you selected to use the User Activity option during set up, press ▲ and ▼ button to select the activity that correlates with your result (refer to page 54), then press ⏻.
- If you do not select to use user activity option during set up, press ⏻ to continue to step 4-2.

### Step 4-2

- If you selected to use the User ID option during setup, press ▲ and ▼ button to select the correct User ID, then press ⏻.

### CAUTION

1. If you did not activate the User Activity option and/or the User ID option, Step 4-1 and/or Step 4-2 procedures are not conducted.
2. Although User Activity option and/or User ID option are activated, if you pull the blood glucose test strip out right after the test, you can not move on to step 4.
3. The test result is not stored under a specific User ID and only displayed after deactivating the User ID option.
3. Please refer to chapter "Setting Your Blood Glucose Test Meter", if you want to set the User Activity option and User ID option.

## Test Strip Ejector Function

1. Once the test is complete, slide the ejector button forward to remove the blood glucose test strip from the blood glucose test meter.

Press and Push forward

Eject your blood glucose test strip

2. Discard the used blood glucose test strip and lancet accordingly to avoid infections.

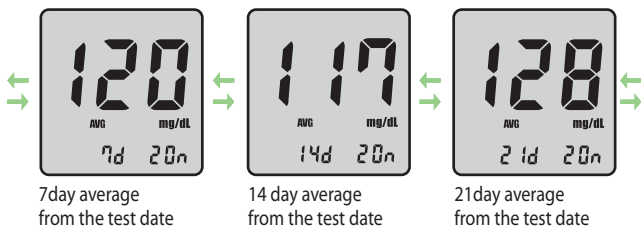
### CAUTION

1. If you push the ejector button forward too much, it may be damaged.
2. Do not apply any force to the control panels.

## Reviewing Your Results

The Glucofine® blood glucose test meter stores up to 400 test results in its built-in memory, along with the average sugar level for the number of days which you had preset the blood glucose test meter to calculate. These values can be recalled in the LCD display at any time.

- Press and release  $\odot$  button. The most recent result is displayed first if user ID feature has not been selected during setting the meter (if ID feature has been selected: see next page).
- Check your previous results by pressing the arrow  $\blacktriangledown$  button.
- Check your average data by pressing the arrow  $\blacktriangle$  button.


## Reviewing Your Results

### Display of average results

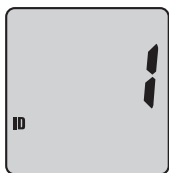- Average test result
- Test Unit
- Test result amount
- Average test result days

### Reviewing results with activated user ID feature:

- Pressing the  button once and the User ID option will appear in the LCD screen.

- After entering the user ID mode, use  and  button to scroll and choose the correct user ID, then press the  button.

- If you choose the User ID to check the stored results, only the results of the selected user are displayed.

## Deleting Test Results

### Deleting individual test result

To delete any individual test result in the memory press ▲ or ▼ button for 3 seconds, while the test result to delete is displayed. **DEL** is blinking. After the third beep sound the test result is deleted.

### Deleting all test results

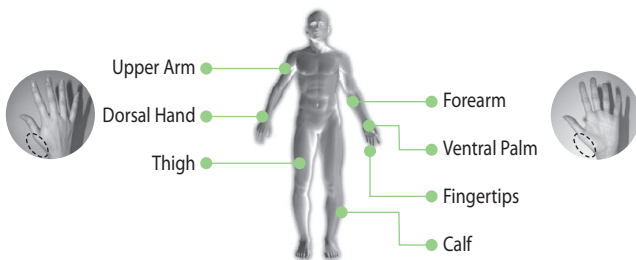
To delete all test results press ▲ and ▼ button at the same time for 3 seconds, while any test result is displayed. **ALL DEL** is blinking.

After the third beep sound all test results are deleted.

### CAUTION

- Only selected Individual test results are deleted when you recalled test results in the user ID condition.
- The deleted test results can't be recovered. Please be careful in deleting the test results.
- If you set the ID function mode, you cannot check the test results of non-ID function mode.
- In order to check the test results of non-ID function mode, please set the ID function mode OFF.

## Alternate Site Testing

### Important Information About Using Alternative Sites Testing :

- Alternate sites where you can test are dorsal hand, ventral palm, upper arm, forearm, calf, and thigh.
- Under certain conditions, blood glucose test results obtained using samples taken from your alternate sites may differ significantly from fingertip samples
- The conditions in which these differences are more likely to occur are when your blood glucose is changing rapidly such as following a meal, as insulin dose, or associated with physical exercise.
- When blood glucose is changing rapidly, fingertip samples show these changes more quickly than alternate sites samples.
- When your blood glucose is falling, testing with a fingertip sample may identify a hypoglycemic (low blood glucose) level sooner than a test with a sample from alternate sites.
- Use alternate sites samples only for testing prior to or more than two hours after meal, insulin doses, or physical exercise.
- Testing performed within two hours after meals, insulin doses, or physical exercise or whenever you feel that your glucose levels may be changing rapidly should be done from the fingertip.
- You should also use fingertip testing whenever you have a concern about hypoglycemia (insulin reactions) such as when driving a car, particularly if you suffer from hypoglycemic unawareness (lack of symptom to indicate as insulin reaction), as forearm testing may fail to detect hypoglycemia.

## Performing Blood Tests in the Alternate Sites

### Important :

We recommend that you test on your fingertips if you are testing for hypoglycemia (low blood glucose) or if you are suffering from hypoglycemia unawareness.

To ensure accurate results when lancing your forearm, upper arm, hand, thigh, or calf, wash the test site with warm clear water.

- Make sure there is no cream or lotion on the test site. Thoroughly dry your hands and test site.

### CAUTION

To receive accurate test results, at least  $0.5\mu\text{l}$  minimum sample volume is required.

## Performing Blood Tests in the Alternate Sites

To bring fresh blood to the surface of the test site, rub the test site vigorously for a few seconds until you feel it getting warm.

Lance the test site with the lancing device:

1. Hold the clear cap down against the top of your test site. Press the release button. Do not lift up.
2. Continue to hold the lancing device and gradually increase pressure for several seconds.
3. While holding the lancing device on your test site, look through the clear cap; until a drop of blood appears approximate • size.
4. Lift the lancing device straight up; be careful not to smear the blood on your testing site.
5. Place the edge of the blood glucose test strip to the drop of blood.
6. The blood will be automatically drawn into the channel of the blood glucose test strip.

# Glucosefine® Software

You can transfer test results from the Glucosefine® blood glucose test meter to a computer.

You can purchase the computer communication cable separately from Metrado GmbH or your distributor.

# Cleaning Your Meter and Maintenance

## Blood glucose test meter :

Your blood glucose test meter does not require special maintenance or cleaning. Avoid getting dirt, dust, blood, glucose control solution, or liquids on the blood glucose test meter, the test port, or data port. Your blood glucose test meter operation temperature is 10-40°C (50-104°F).

It is recommended that you store the blood glucose test meter in its carrying case after each use.

A cloth dampened with water and mild detergent can be used to wipe down the outside of the blood glucose test meter. Your Glucofine® blood glucose test meter is a precision instrument. Please handle it with care.

## Lancing device :

Clean the lancing device and caps with soap and tepid water. To disinfect the lancing device, prepare a disinfectant solution of one part household bleach to 10 parts water. Dampen a cloth with this solution and wipe the lancing device thoroughly. Soak only the cap for at least 30 minutes in the disinfectant solution. Do not soak the lancing device in liquid. Rinse the lancing device and cap with water and dry thoroughly.

- For detailed instructions concerning Glucofine® blood glucose test strips, refer to the blood glucose test strip instruction leaflet found in the blood glucose test strip box.
- Please store the Glucofine® blood glucose measuring system in a cool and dry area, out of reach of children. Do not freeze. For a more accurate test result avoid testing under direct sunlight.
- Do not soak the blood glucose test meter or blood glucose test strips in water or liquid.
- Do not subject the meter or blood glucose test strips to excessive heat.
- Use your meter according to the instructions in the manual.
- If you need to purchase the lancing device, lancets or blood glucose test strips, contact your pharmacy or local representative.
- Discard the used lancets carefully, to prevent any infection.
- The blood glucose test meter could be sent back to your local representative for recycling or proper disposal. Alternatively the meter shall be disposed in accordance with national laws.

## Replacing the Batteries

The low battery icon will appear in the upper left corner of the LCD screen to alert you when the battery power is running low, indicating a new battery is needed.

Your Glucofine® blood glucose test meter uses only two 3V Lithium batteries (CR2032), which are included. When replacing the batteries, only CR2032 or equivalent lithium battery should be used.

Before replacing the batteries, make sure your meter is turned off. When changing batteries, match the polarity symbols printed on the battery. Otherwise the meter will not work!

### <Follow-up Action>

1. Check your meter with the glucose control solution.
2. Perform your test again.
3. If you are experiencing symptoms that are not consistent with your blood glucose test results, call your healthcare professional.

### CAUTION

Remove both batteries from the battery compartment and dispose of them according to your institution's guidelines.

# Troubleshooting

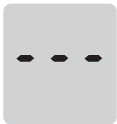
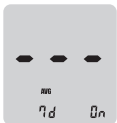
The following chart may help you identify certain problems, but may not solve all problems that can occur. Contact your authorized representative if the problem persists.

Message	Problem	Action Required
	Problem with the blood glucose test meter	Place the battery again, and set the blood glucose test meter. If the problem persists, please contact your authorized representative or customer support.
	Caused by either used or wet blood glucose test strip	Please insert a new blood glucose test strip and perform your test again
	Problem with the test strip	Messung mit neuem Teststreifen wiederholen und solange Blut auftragen, bis das Fenster komplett gefüllt ist.
	Blood glucose test strip is damaged	Please insert a new blood glucose test strip and perform your test again
	User applied the blood sample before the LCD display  was blinking	Please wait for the LCD to display icon blinking before applying your blood sample

## Troubleshooting

Message	Problem	Action Required
	Problem with the Auto-coding label	Please insert a new test strip and perform your test again. If Er6 message persists, please contact your local representative. If blink with "Sun", avoid to the direct sunlight and retest.
	The ambient temperature is too low	Place the blood glucose test meter within the operating temperature range for 10 minutes or more and retest.
	The ambient temperature is too high	Place the meter within the operating temperature range for 10 minutes or more and retest.
	The test result is lower than 10mg/dL	Check the accuracy of your blood glucose test strip by performing a glucose control solution test. If the test results in a normal reading, re-test your blood sample two or three times. If "Lo" persists, consult your doctor immediately.
	The test result is higher than 600mg/dL	Check the accuracy of your blood glucose test strip by performing a glucose control solution test. If the test results in a normal reading, re-test your blood sample two or three times. If "Hi" persists, consult your doctor immediately.

# Troubleshooting

Message	Problem	Action Required
The blood glucose test meter does not power on	Battery is dead or there is a problem with the blood glucose test meter	Change the battery and if the problem persists, contact your local representative
Test does not start after applying blood sample	Poor blood sample	Please insert a new blood glucose test strip and perform your test again
The result is inconsistent	There may be a problem with the blood glucose test strip	Please insert a new blood glucose test strip and perform your test again
	No readings in memory to display	
	No enough readings in memory to display designated averages	

## Specifications

Model Name	Glucosefine®
Test Range	10 - 600 mg/dL ( 0.6 - 33.3 mmol/L )
Reading time	5 seconds
Memory Capacity	400 test results
Operating Altitude	up to 10000 feet (3000 meters)
Operating Temperature	10-40 °C (50-104°F)
Operating Humidity	10 - 90 %
Sample Type	Capillary Whole Blood
Sample Volume	0.5 $\mu$ l
Calibration	Plasma-calibrated
Display Type	Backlight LCD
Size (WxLxH)	50.3 x 88.4 x 18.5 mm
Weight	46 g (including batteries)
Power Supply	3V (Coin Battery, CR2032 X 2EA)
Battery Life	Running 1,000 tests

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# Warranty

## Three - Year Warranty

If, at any time during the first three years after purchase, the blood glucose test meter does not work for any reason with the exception of obvious abuse, misuse, or disastrous damage, we will replace your Glucofine® blood glucose test meter or equivalence free of charge.

The Warranty Policy only applies to the original purchaser of the blood glucose test meter and will not include the batteries supplied with the blood glucose test meter set.

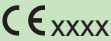
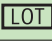
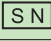
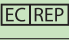
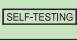
Please fill in the warranty card attached to the set.  
Mail or fax it back to METRADO GmbH.

The Glucofine® blood glucose test meter has a full three-year warranty from the original date of your purchase. For your convenience, please include your date and place of purchase.

METRADO GmbH  
Auf Pfuhlst 1  
D-66589 Merchweiler  
GERMANY  
Tel. +49 (0)6825 95 22 78 -0  
Fax. +49 (0)6825 95 22 78 -1  
[www.metrado.de](http://www.metrado.de)

## Symbol Reference

Symbol	Description
	Consult operating instructions
	Expiration date
	Discard 3 months after opening
	This product fulfills the requirements of Directive 98/79/EC on in vitro diagnostic medical devices
	This product meets the requirements of the Regulation (EU) 2017/745
	Attention, see instructions for use
	Medical device
	For in vitro diagnostic use
	Batch code
	Catalogue Number
	Temperature limitation
	Do not reuse
	Serial number
	Date of Manufacture
	Manufacturer
	EU authorized representative for medical devices
	Keep away from sunlight
	Self-testing Used
	Discard batteries and electronic equipment according to local regulations. Do not dispose with household waste.

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**SteriLance Medical (Suzhou) Inc.**
METRADO GmbH, Auf Pfuhlst 1, 66589 Merchweiler, Germany, [www.metrado.de](http://www.metrado.de)